

DP.Universal Fill

Light-curing Composite Resins

Teeth Renewed. Strength Enduring.



Flowable
Restorative

Universal
Restorative

DEPRAG DP.Universal Fill: Newly Launched Light-Cure Restorative Resin

DEPRAG's latest innovation, DP.Universal Fill, is a versatile dental restorative composite resin. Offered in two optimized consistencies:

Universal Resin: Enables quick and easy sculpting of tooth anatomy.

Ideal for:

- ⊕ Direct anterior restorations
- ⊕ Splinting
- ⊕ Core buildups
- ⊕ Indirect restorations

Flowable Resin: Effortlessly penetrates and adapts to fine details and intricate areas.

Perfect for:

- ⊕ Class III & V cavity restorations
- ⊕ Lining/basing in direct restorations
- ⊕ Minimally invasive cavity preparations
- ⊕ Pit and fissure sealing
- ⊕ Undercut blocking



Combine both for optimal esthetics and clinical efficiency.

Flowable Resin

- 👑 Exceptional Esthetics;
- 👑 Excellent Flowability for Sealing Fine Details;
- 👑 Enduring Performance;



Universal Resin

- 👑 Natural-Like Luster;
- 👑 Effortless Application;
- 👑 Exceptional Strength & Wear Resistance;



Highlight! Reliable resin material.



Superior Materials Deliver Superior Performance.

German-Engineered Materials: Featuring superior handling characteristics; high post-cure strength; and long-lasting restorations.



Superior Esthetics: The Leading Reason to Choose It.

Featuring inherently superior polish retention, natural fluorescence, and enamel-like luster and translucency post-curing – delivering the closest match to natural dentition.



Rapid Cure.

Achieves full polymerization in just 20 seconds, boosting clinical efficiency.



Comprehensive Shade Selection

Available in VITA Classical shades:
A1, A2, A3, A3.5 - delivering versatile coverage for natural-looking clinical results.




4 Steps to Natural Esthetic Results:

STEP 01 Shade Selection & Verification




Confirm shade match pre-cure using trial specimens under natural light.

STEP 02 Tooth Preparation

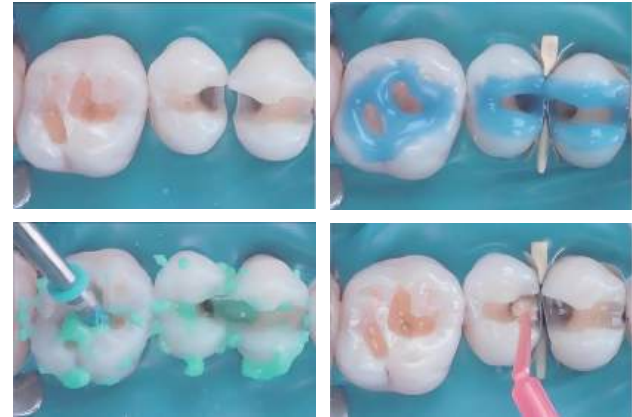


Perform isolation, cavity preparation, etching, bonding agent application, and light-curing per manufacturer protocol.

STEP 03 Layered Resin Application

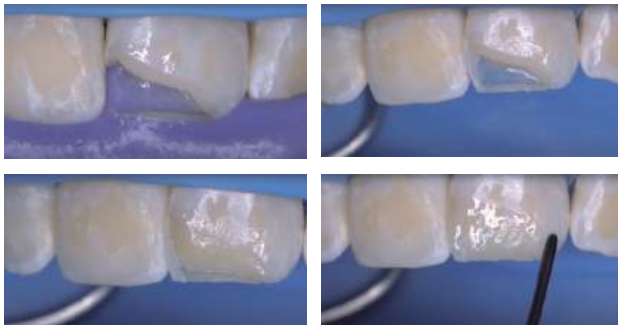

Place material in ≤ 2 mm increments; light-cure each layer for ≥ 20 seconds using validated curing lights

STEP 04 Finishing & Polishing

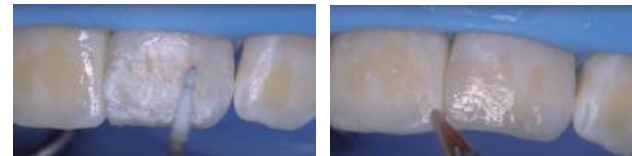

Refine anatomy with coarse diamonds → Finalize with fine-grit polishing systems → Achieve enamel-like surface texture.

Case Showcase

Aesthetic shaping of anterior teeth



As a Class V Restoration



As a Class IV Restoration



ABOUT US

DEPRAG is a global dental technology enterprise specializing in digital dental materials, smart devices, and cloud-based services, with comprehensive coverage across restorative, orthodontic, and implant dentistry. Through material science innovation and digital ecosystem development, the company is driving the transformation of dental care from traditional experience-driven practices to a digitally-driven model.

Zhengzhou

Headquarters
Dental Materials and Software R&D Center
Marketing Center
Dental Lab

Shenzhen

Dental Equipment R&D, Manufacturing and Training Center
Product Display Center

Anyang

Dental Materials Manufacturing Center



- 25+ Excellent dealer
- 1000+ Employees
- 90+ Countries and Regions

DEPRAG (Zhengzhou) Dental Technology Co., Ltd.

Web: depragdental.com

Email: info@depragdental.com

Add: No.14, No.8 Floor, Building 7, No.289 West Third Ring Road, High-tech Industrial Development Zone, Zhengzhou City, Henan Province, China.

